

DSSE

PRODUCT MANUAL



CHANIA GOLD STOOL MS16

Channelled Beauty for Modern Spaces

The Chania Stool has a light, architectural presence that feels intentionally designed rather than purely functional. Its slim champagne gold frame and channelled seat create a refined, contemporary silhouette, while generous cushioning keeps it comfortable through long working days. Finished in antimicrobial medical grade vinyl and set on a five star base with smooth gas-lift control, Chania moves easily with you, balancing understated style with practical performance in any salon or clinic.

Disclaimer

This manual is provided solely as a reference for the safe and proper operation of this model and must not be used for any other purpose. DSSE accepts no liability for any incidental, indirect, or consequential damages arising from the use of this manual or the equipment described.

The information contained herein is subject to change without notice and does not constitute a commitment by DSSE. While every effort has been made to ensure accuracy, DSSE assumes no responsibility for errors or omissions, nor any expressed or implied warranties, including but not limited to merchantability or fitness for a particular purpose. All content is protected by copyright law and may not be copied, reproduced, distributed, or translated, in whole or in part, without prior written authorisation from DSSE.

General Cleaning

- Routine care – Dust regularly with a soft, dry cloth to prevent buildup.
- Mild cleaning – Use a damp cloth with mild soapy water for most surfaces. Wipe dry immediately with a clean cloth.
- Avoid abrasive materials or harsh cleaners, as they may damage the surface.
- Test first – Always test any new cleaning product on a small, hidden area before full use.

Safety Information

- To ensure safe use of this product, please observe the following precautions:
Keep children away from the equipment; place it out of their reach.
- Use only as intended – The product must only be used for its designed purpose. Misuse may result in injury or damage.
- Weight capacity – Do not exceed the recommended weight limit. Overloading may cause product failure or accidents.
- Stable placement – Use only on a flat, level surface.
- Assembly and maintenance – Ensure all parts are correctly assembled and regularly check that fasteners are secure.
- Use the stool only for sitting. Do not stand, lean excessively, or use it as a step. Avoid rolling around the room or using it in a skateboard-like manner, as this can cause loss of balance or injury.
- Professional use only – This product is intended for professional salon/clinic environments and should only be operated by trained personnel.

Maintenance

- Periodically check and gently tighten all screws, bolts, and fittings to ensure long-term stability.
- Replace any damaged, worn, or missing components immediately.
- Inspect the gas lift and base to ensure they are stable and functioning correctly.
- Keep the wheels clean and free of hair or debris; replace any worn or damaged wheels promptly for optimal movement.
- Wipe the seat and base with a mild cleaning solution and soft cloth to maintain appearance—avoid abrasive or harsh chemicals.
- All maintenance should be carried out by a suitably qualified technician or tradesperson.
- Use only manufacturer-approved replacement parts like gaslifts and wheels to maintain product safety and warranty coverage.

WARRANTY PERIOD

STANDARD: 1 YEAR | VINYL: 2 YEARS

For more information about our warranty, please refer to our website.

Product Specifications

Height:	78-94cm
Seat Height:	51-67cm
Width:	52cm
Depth:	52cm
Backrest Height from Seat:	30cm
Backrest Width:	43cm
Base Diameter:	52cm
Net Weight:	14 kg
Gross Weight:	10 kg
Maximum Weight Capacity:	120 kg Seating / 20 kg Backrest




Installation Preparation: Before assembly, carefully remove all parts from the packaging and verify that all components are included. Place the stool on a clean, flat surface to prevent damage during installation. It is recommended to have one person perform the assembly. Inspect all parts for any transit damage before use.




STEP 1

Turn the 5-star base (D) upside down and insert the five castors (B) firmly into each leg socket. Apply downward pressure until each castor clicks securely into place.

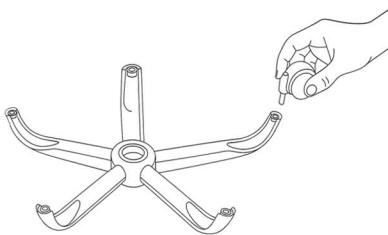


STEP 2

Turn the base upright on a flat, stable surface. Insert the gas lift (E) into the center hole of the aluminium base, ensuring it sits straight and firmly in position.

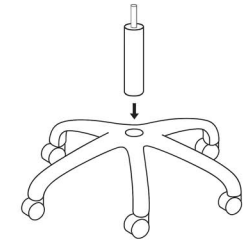

STEP 3

Loosen and remove the screws on both sides of the seat (A). Align one armrest (D) with the mounting holes, then reinstall the screws and tighten securely. Repeat this process for the backrest, and then follow the same steps on the opposite side.

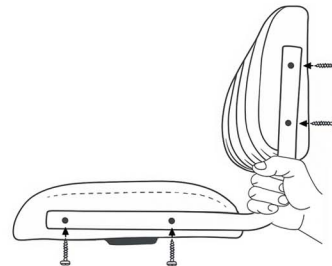

STEP 4

Align the seat with the top of the gas lift (F), then press and hold the height adjustment lever upward while gently lowering the seat to lock the mechanism securely into place.



SAFETY WARNING: This product is designed for professional use only. Ensure it is placed on a stable, level surface and never exceed the stated weight capacity of 120 kg. Children should not sit, play, or operate the stool at any time. The stool is not intended to be rolled or moved at high speed while seated.