

# DSSE

## PRODUCT MANUAL



## COSMEDICA WITH INCLINE TB004-INC

---

### **The Ultimate Treatment Bed for Skin Injectables**

The Cosmedica Treatment Chair offers sleek design and premium functionality with four adjustable motors for height, backrest, footrest, and tilt. Featuring reversible, removable armrests and a vertical pedestal base, it's perfect for compact spaces. The tapered base allows therapists to get closer for precise treatments. With its minimalist design and exceptional features, the Cosmedica is the ideal choice for any spa or salon seeking to provide a luxurious client experience.

## Introduction

This manual is intended solely as a reference for the operation of this specific model. DSSE assumes no responsibility or liability for any consequences resulting from its use for other purposes. All information contained in this manual is protected by copyright and may not be copied, reproduced, or translated, in whole or in part, without the prior written consent of our company.

### Safety Instructions

- Always disconnect the power supply before changing electrical components or performing maintenance and cleaning on the equipment.
- The plug must be connected to a properly installed and grounded outlet to prevent the risk of fire or electric shock.
- Do not stand on the equipment.
- Keep children away from the equipment; place it out of their reach. Always adjust the backrest to its highest position when leaving the equipment unattended.
- Do not place the equipment in damp or wet areas such as bathrooms to avoid electric shock or malfunction. The remote control should also be kept away from water.
- Avoid using the equipment in environments where painting, coating, or oxygen provision is taking place.
- Unless supervised or instructed by qualified personnel, people with physical disabilities, sensory disorders, mental impairments, lack of experience or common sense, and children are not permitted to use the equipment.
- If the plug is damaged, contact your distributor, service center, or a qualified professional for repair.
- Ensure there are no objects within the equipment's movement range during operation.
- All maintenance must be performed by trained professionals only.

### Precautions

- The equipment should be operated by experienced personnel familiar with its structure, performance, and instructions.
- Read this manual carefully before installation, operation, and adjustment. Keep it for future reference.
- Do not use for single-side or single-point loading.
- Follow safety operation rules. Do not overload. Ensure all safety equipment is reliable and address any hazards immediately.
- Regularly tighten bolts and nuts to avoid failure due to loosening or vibration.
- Do not disassemble parts without proper authorization.
- If faults occur or abnormal noises are heard, stop usage immediately and have the equipment inspected.
- Only resume use after proper troubleshooting.
- This bed uses a three-wire power cord and must be plugged into a grounded outlet.
- Observe all warning labels on the product.
- Press and hold the control button when adjusting the bed height until the desired position is reached.
- Motor operation time: 2 minutes ON, 18 minutes OFF to prevent overheating).
- Clean dust and stains using a neutral cleaning agent after each treatment.
- Keep hands, feet, and head clear of moving parts during adjustments.

# PRECISION COMFORT, START INJECTING.

A fan favorite, the Cosmedica Treatment Chair is the perfect choice for cosmetic injectables and upright chair treatments. Upholstered in ultra-soft, luxurious materials, it provides exceptional comfort and style, making it a standout in any high-end clinic or salon. The aesthetically pleasing slimline base and tower offer a modern, space-saving design without compromising functionality. Its curved backrest and legrest design enable up-close, precise treatments, ensuring a seamless experience for both clients and practitioners. The Cosmedica is the ideal treatment chair for any professional clinic, combining luxury, performance, and style.

### Adjustable Backrest (0° - 85°)

Easily adjustable from 0° to 80°, the backrest allows you to customize the seating angle for optimal comfort and support.

### Height Adjustment (63 - 83 cm)

Adjustable height for ergonomic comfort and ease of use.

### Hand Remote Clip

A convenient clip is located on either side of the bed to securely hang your hand remote, keeping it within easy reach during treatments.

### Slimline Base

The sleek, tapered base design allows for up-close treatments, providing both space-saving functionality and easy access for precise work.

### Adjustable (pull-out) headrest (18cm)

The pull-out headrest provides adjustable support for enhanced comfort, allowing you to extend or retract as needed to perfectly accommodate different clients and treatment positions.

### Swiveling Armrests (0°-180°)

Designed for maximum comfort and flexibility, the swiveling armrests easily rotate to provide optimal support and allow effortless access during treatments.

### Adjustable Legrest (0°-45°)

Effortless adjustment for maximum comfort and support.

### Tilt Function (0°-11°)

The 4th motor tilt function ensures comfort and safety for fainting clients or specialized treatments.



## Product Specifications

Length:	<b>193cm</b>
Width without/with Arms:	<b>56/84cm</b>
Height Range:	<b>63-83cm</b>
Backrest Adjustment Angle:	<b>Up to 85°</b>
Seat Tilt Adjustment Angle:	<b>Up to 11°</b>
Legrest Adjustment Angle:	<b>Up to 45°</b>
Net Weight:	<b>70 kg</b>
Gross Weight:	<b>83 kg</b>
Max. Load Capacity:	<b>180 kg (seating position)</b>
Power Supply:	<b>AC 220-240V, 50Hz/60Hz</b>
Max. Power Consumption:	<b>300W</b>




## Effortless Hand Remote Control

The Cosmedica treatment bed is equipped with an intuitive hand-held remote (optional foot remote available) that allows effortless adjustment of height, backrest, and leg sections. Its responsive soft-touch buttons provide smooth, precise positioning to suit your working needs—ensuring uninterrupted focus during treatments and enhanced ergonomic comfort for every professional.

## How To Use

Follow these instructions to operate the electric beauty bed safely and effectively.

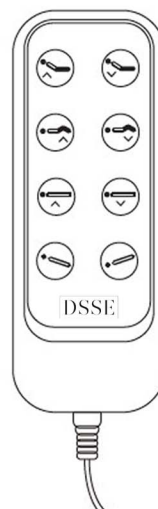

### Backrest Adjustment

Press to raise or lower the backrest



### Legrest Adjustment

Press to raise or lower the legrest



### Height Adjustment

Press to raise or lower the bed



### Incline Adjustment

Press to tilt (incline) the bed

## Upholstery Care Guide

As a professional esthetician, investing in a high-quality electric facial bed can significantly enhance the comfort and effectiveness of your treatments. To maintain that level of performance and ensure long-term reliability, regular maintenance and proper care are essential. Like any professional equipment, your facial bed requires routine attention to stay in optimal working condition—ensuring safety, smooth functionality, and a consistently elevated experience for your clients.

### General Care Tips

Maintain a stable temperature:

- Cold can lead to vinyl cracking
- Heat may cause permanent stretching

Keep out of direct sunlight for long periods to prevent fading and damage. Regular housekeeping helps preserve comfort and appearance.



### Cleaning Instructions

Clean frequently to remove body oils that can harden vinyl

Use a colorfast cloth dampened with lukewarm water and mild, alkaline-free soap

Avoid soaking the material

Wipe off any soap with fresh water and a clean cloth

Dry thoroughly with a soft, dry cloth

For tougher grime, gently scrub with a soft brush



## Avoid Using These Products

- Hot or boiling water
- Alcohol or hydrocarbon-based cleaners
- Acetone
- Bleach or harsh detergents
- Furniture polish
- Abrasive powders, pastes, or steel wool
- Silicone or petroleum-based products
- Solvent-based or abrasive cleaning agents

## Routine Inspection

Perform periodic checks on:

- Hinges
- Fasteners
- Structural components

Ensure your chair remains safe, stable, and fully functional.

Do not dispose of old or defective appliances with household waste. Please use designated public collection points for proper disposal. This manual is for reference only. Specifications may change without prior notice.

DSSE

Australia:

[www.dsse.com.au](http://www.dsse.com.au)

United Kingdom:

[www.dssebeauty.co.uk](http://www.dssebeauty.co.uk)

Europe:

[www.dssebeauty.eu](http://www.dssebeauty.eu)