

DSSE

PRODUCT MANUAL



ELECTRIC BED 3 MOTOR BB08

Seamless Functionality Meets Customised Comfort

Engineered for performance and designed for versatility, the Electric Bed 3 Motor offers precision positioning and outstanding client comfort. This 3-piece treatment couch is equipped with a powerful 3-motor system that allows smooth, quiet adjustment of height, backrest, and leg angle—ideal for a wide range of therapies and aesthetic treatments.

Introduction

This manual is intended solely as a reference for the operation of this specific model. DSSE assumes no responsibility or liability for any consequences resulting from its use for other purposes. All information contained in this manual is protected by copyright and may not be copied, reproduced, or translated, in whole or in part, without the prior written consent of our company.

Safety Instructions

- Always disconnect the power supply before changing electrical components or performing maintenance and cleaning on the equipment.
- The plug must be connected to a properly installed and grounded outlet to prevent the risk of fire or electric shock.
- Do not stand on the equipment.
- Keep children away from the equipment; place it out of their reach. Always adjust the backrest to its highest position when leaving the equipment unattended.
- Do not place the equipment in damp or wet areas such as bathrooms to avoid electric shock or malfunction. The remote control should also be kept away from water.
- Avoid using the equipment in environments where painting, coating, or oxygen provision is taking place.
- Unless supervised or instructed by qualified personnel, people with physical disabilities, sensory disorders, mental impairments, lack of experience or common sense, and children are not permitted to use the equipment.
- If the plug is damaged, contact your distributor, service center, or a qualified professional for repair.
- Ensure there are no objects within the equipment's movement range during operation.
- All maintenance must be performed by trained professionals only.

Precautions

- The equipment should be operated by experienced personnel familiar with its structure, performance, and instructions.
- Read this manual carefully before installation, operation, and adjustment. Keep it for future reference.
- Do not use for single-side or single-point loading.
- Follow safety operation rules. Do not overload. Ensure all safety equipment is reliable and address any hazards immediately.
- Regularly tighten bolts and nuts to avoid failure due to loosening or vibration.
- Do not disassemble parts without proper authorization.
- If faults occur or abnormal noises are heard, stop usage immediately and have the equipment inspected.
- Only resume use after proper troubleshooting.
- This bed uses a three-wire power cord and must be plugged into a grounded outlet.
- Observe all warning labels on the product.
- Press and hold the control button when adjusting the bed height until the desired position is reached.
- Motor operation time: 2 minutes ON, 18 minutes OFF to prevent overheating).
- Clean dust and stains using a neutral cleaning agent after each treatment.
- Keep hands, feet, and head clear of moving parts during adjustments.

EFFORTLESS ELEGANCE, EVERYDAY PERFORMANCE

Experience supreme luxury with this beautifully crafted treatment bed, designed to deliver exceptional comfort and support for your clients. Ideal for all types of skin treatments, its plush upholstery and aesthetically refined base add a sophisticated touch to any salon space. Equipped with three silent motors, it offers smooth, whisper-quiet adjustments that enhance both client comfort and professional efficiency.

A seamless fusion of style and functionality, the 3 Motor Electric Bed elevates every treatment into a truly effortless experience.

Adjustable Backrest (0° - 77°)

Easily adjustable from 0° to 77°, the backrest allows you to customize the seating angle for optimal comfort and support.

Height Adjustment (62 - 93cm)

Adjustable height for ergonomic comfort and ease of use.

Power Source

Power board located beneath the head section at the rear, featuring an integrated on/off switch and IEC socket for easy access

Elegance from the Ground Up

Sleek, slimline base offers enhanced access for detailed treatments.

Curved Design

Designed with a soft curved edge to bring you closer to your client

Padded Armrests

Removable armrests provide added comfort and support, while allowing flexibility for closer access during treatments.

High Quality Materials

Upholstered in premium medical-grade vinyl and layered with high-density memory foam, offering exceptional comfort and long-lasting support in every treatment.


Adjustable Legrest (0°-50°)

Effortless adjustment for maximum comfort and support.



Product Specifications

Length:	186cm
Width without/with Arms:	62/83cm
Height Range:	60-83cm
Backrest Adjustment Angle:	Up to 77°
Legrest Adjustment Angle:	Up to 50°
Net Weight:	72 kg
Gross Weight:	77 kg
Max. Load Capacity:	180 kg (seating position)
Power Supply:	AC 220-240V, 50Hz/60Hz
Max. Power Consumption:	300W

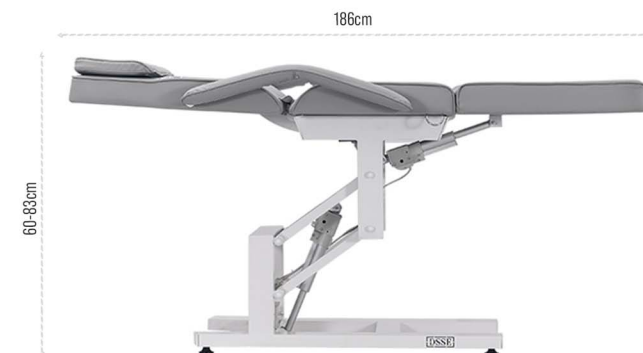

Effortless Hand Remote Control

The Electric Bed 3 Motor comes equipped with an intuitive hand-held remote, with an optional foot control available for hands-free operation. Its responsive soft-touch buttons allow effortless adjustment of the height, backrest, and leg sections—ensuring smooth, precise positioning for optimal working posture and uninterrupted treatment flow. Designed to enhance ergonomic comfort, it supports professionals in delivering focused, efficient care.

How To Use

Follow these instructions to operate the electric beauty bed safely and effectively.



Backrest Adjustment

Press to raise or lower the backrest



Legrest Adjustment

Press to raise or lower the legrest



Height Adjustment

Press to raise or lower the bed

Upholstery Care Guide

As a professional esthetician, investing in a high-quality electric facial bed can significantly enhance the comfort and effectiveness of your treatments. To maintain that level of performance and ensure long-term reliability, regular maintenance and proper care are essential. Like any professional equipment, your facial bed requires routine attention to stay in optimal working condition—ensuring safety, smooth functionality, and a consistently elevated experience for your clients.

General Care Tips

Maintain a stable temperature:

- Cold can lead to vinyl cracking
- Heat may cause permanent stretching

Keep out of direct sunlight for long periods to prevent fading and damage. Regular housekeeping helps preserve comfort and appearance.



Cleaning Instructions

Clean frequently to remove body oils that can harden vinyl

Use a colorfast cloth dampened with lukewarm water and mild, alkaline-free soap

Avoid soaking the material

Wipe off any soap with fresh water and a clean cloth

Dry thoroughly with a soft, dry cloth

For tougher grime, gently scrub with a soft brush



Avoid Using These Products

- Hot or boiling water
- Alcohol or hydrocarbon-based cleaners
- Acetone
- Bleach or harsh detergents
- Furniture polish
- Abrasive powders, pastes, or steel wool
- Silicone or petroleum-based products
- Solvent-based or abrasive cleaning agents

Routine Inspection

Perform periodic checks on:

- Hinges • Fasteners • Structural components

Ensure your chair remains safe, stable, and fully functional.

Do not dispose of old or defective appliances with household waste. Please use designated public collection points for proper disposal. This manual is for reference only. Specifications may change without prior notice.

DSSE

Australia: www.dsse.com.au
United Kingdom: www.dssebeauty.co.uk
Europe: www.dssebeauty.eu