

DSSE

PRODUCT MANUAL



GALEA TREATMENT BED TB021

Turn with Total Control - Every Angle, Every Treatment

The Galea Treatment Bed is built for intelligent movement and adaptable positioning in professional treatment spaces. Its four-motor system delivers smooth control of backrest, height, legrest, and incline, while the signature 210-degree rotation enhances access and workflow. Side backrest and seat controls with easy to reach buttons ensure fast, repeatable adjustments. Upholstered in antimicrobial vinyl with plush cushioning and a streamlined enclosed base, Galea combines precision, comfort, and modern design.

Introduction

This manual is intended solely as a reference for the operation of this specific model. DSSE assumes no responsibility or liability for any consequences resulting from its use for other purposes. All information contained in this manual is protected by copyright and may not be copied, reproduced, or translated, in whole or in part, without the prior written consent of our company.

Safety Instructions

- Always disconnect the power supply before changing electrical components or performing maintenance and cleaning on the equipment.
- The plug must be connected to a properly installed and grounded outlet to prevent the risk of fire or electric shock.
- Do not stand on the equipment.
- Keep children away from the equipment; place it out of their reach. Always adjust the backrest to its highest position when leaving the equipment unattended.
- Do not place the equipment in damp or wet areas such as bathrooms to avoid electric shock or malfunction. The remote control should also be kept away from water.
- Avoid using the equipment in environments where painting, coating, or oxygen provision is taking place.
- Unless supervised or instructed by qualified personnel, people with physical disabilities, sensory disorders, mental impairments, lack of experience or common sense, and children are not permitted to use the equipment.
- If the plug is damaged, contact your distributor, service center, or a qualified professional for repair.
- Ensure there are no objects within the equipment's movement range during operation.
- All maintenance must be performed by trained professionals only.

Precautions

- The equipment should be operated by experienced personnel familiar with its structure, performance, and instructions.
- Read this manual carefully before installation, operation, and adjustment. Keep it for future reference.
- Do not use for single-side or single-point loading.
- Follow safety operation rules. Do not overload. Ensure all safety equipment is reliable and address any hazards immediately.
- Regularly tighten bolts and nuts to avoid failure due to loosening or vibration.
- Do not disassemble parts without proper authorization.
- If faults occur or abnormal noises are heard, stop usage immediately and have the equipment inspected.
- Only resume use after proper troubleshooting.
- This bed uses a power cord and must be plugged into a grounded outlet.
- Observe all warning labels on the product.
- Clean dust and stains using a neutral cleaning agent after each treatment.
- Keep hands, feet, and head clear of moving parts during adjustments.
- Always observe the maximum user weight capacity of the bed.
- Do not sit or apply heavy force on armrests, head or back sections. Refer to the weight limits in this manual.
- Motor operation time: 2 minutes ON, 18 minutes OFF to prevent overheating).
- Clients should enter and exit the bed from the middle section only.

MEASURED MOTION, PROFESSIONAL MASTERY.

Galea is engineered for intelligent movement, smooth rotation, and reliable performance in professional practice. Its four-motor system provides precise control of backrest, height, legrest, and incline, allowing practitioners to position clients effortlessly for a wide range of treatments.

The standout 210-degree rotation enables the bed to pivot 105 degrees each way from centre, improving access, visibility, and workflow without moving the base. Side backrest and seat controls allow for easy adjustments, while a reset button on the backrest quickly returns the bed to its original position at the touch of a button.

Upholstered in antimicrobial vinyl with plush cushioning and finished with a clean enclosed base, Galea delivers stability, hygiene, and everyday dependability across beauty, brow, lash, and spa services.

Adjustable & Tilttable (Pull-out) Headrest (13cm)

The pull-out headrest extends for added support and tilts for flexible positioning during treatments.

Adjustable Backrest (0° - 85°)

Easily adjustable from 0° to 85°, the backrest allows you to customize the seating angle for optimal comfort and support.

Integrated Backrest & Seat Controls

Designed for quick and easy adjustments during treatments.

Rotatable (±105°)

Smooth rotation up to 105° each way (from centre) for ultimate flexibility and ease.

Height Adjustment (65- 98 cm)

Adjustable height for ergonomic comfort and ease of use.

Removable & Swiveling Armrests (0°-180°)

The armrests swivel smoothly from 0° to 180° and can be removed as required, providing flexible positioning, comfortable support, and easy access during treatments or procedures.

Inclineable Seat (10°)

Enjoy personalized comfort with a seat that gently tilts up to 10°.

Adjustable Legrest (0°-90°)

Effortless adjustment for maximum comfort and support.

Pull-Out Footrest

Easily extends to support legs, enhancing comfort during use.

Heavy Duty Base

Solid, reliable, and designed to withstand high-volume daily use.



Product Specifications

Length:	189cm
Width without/with Arms:	60cm/85cm
Height Range:	65-98cm
Backrest Adjustment Angle:	Up to 85°
Legrest Adjustment Angle:	Up to 90°
Seat Tilt Adjustment Angle:	Up to 10°
Net Weight:	95kg
Gross Weight:	102kg
Max. Load Capacity:	200 kg (seating position)
Power Supply:	AC 220-240V, 50Hz/60Hz
Max. Power Consumption:	300W




Effortless Button Operation

Strategically positioned on the side of the backrest and discreetly integrated into the base, the Effortless Button offers quick, intuitive access to all bed adjustments.


With a simple press, the bed moves smoothly and precisely into your ideal working position, allowing you to maintain focus without interruption.

Designed with professional beauticians in mind, the responsive, ergonomic buttons enable seamless adjustments with minimal physical effort, supporting better posture, reducing strain, and enhancing comfort throughout long treatment sessions.

Backrest Cover Buttons

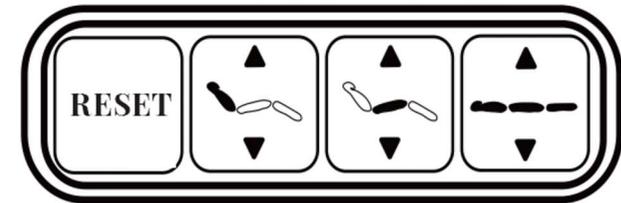

Inbuilt Base Buttons

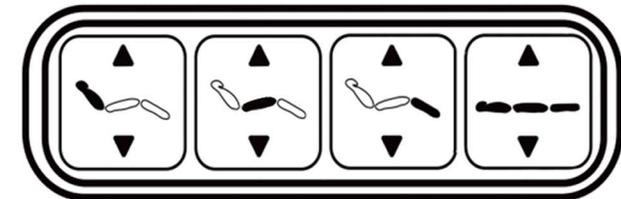


HOW TO USE

Follow these instructions to operate the electric beauty bed safely and effectively. Ensure all users are familiar with the controls and functions before use.



Backrest Adjustment

Press to raise or lower the backrest

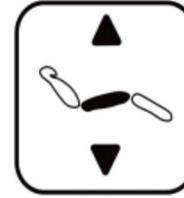

Incline Adjustment

to tilt the bed into the Trendelenburg position for inclined treatments



Height Adjustment

Press to raise or lower the bed between 59-94 cm



Legrest Adjustment

to raise or lower the legrest between 0° and 85°



Armrest Adjustment

Rotate armrests 180° for full access and flexibility.




Rotation Adjustment

Unlock the handle to rotate the chair 105° left or right. Push it back in to securely lock it in place.



Hand Remote Operation


- 1** Press button to raise the backrest.
- 2** Press button to lower the backrest.
- 3** Press button to raise the incline.
- 4** Press button to lower the incline.
- 5** Press button to raise the legrest.
- 6** Press button to lower the legrest.
- 7** Press button to raise the height.
- 8** Press button to lower the height.
- 9** Press and hold the button for three seconds to reset the chair. The backrest will recline fully, the chair will lower to its minimum height, the seat cushion will tilt to its maximum position, and the leg rest will automatically return to its lowest position.
- 10** Press button to disable (lock) all functions. Press it again to unlock and reactivate the controls.
- 11** Memory Function - Hold the button for 3 seconds until it activates. fully, the chair will lower to Then press **1** to save the bed in that position. You can save other positions by doing the same with **2** or **3**.
- 12** Hold the button for 3 seconds to move the chair automatically to Memory Position **P1**.
- 13** Hold the button for 3 seconds to move the chair automatically to Memory Position **P2**.
- 14** Hold the button for 3 seconds to move the chair automatically to Memory Position **P3**.

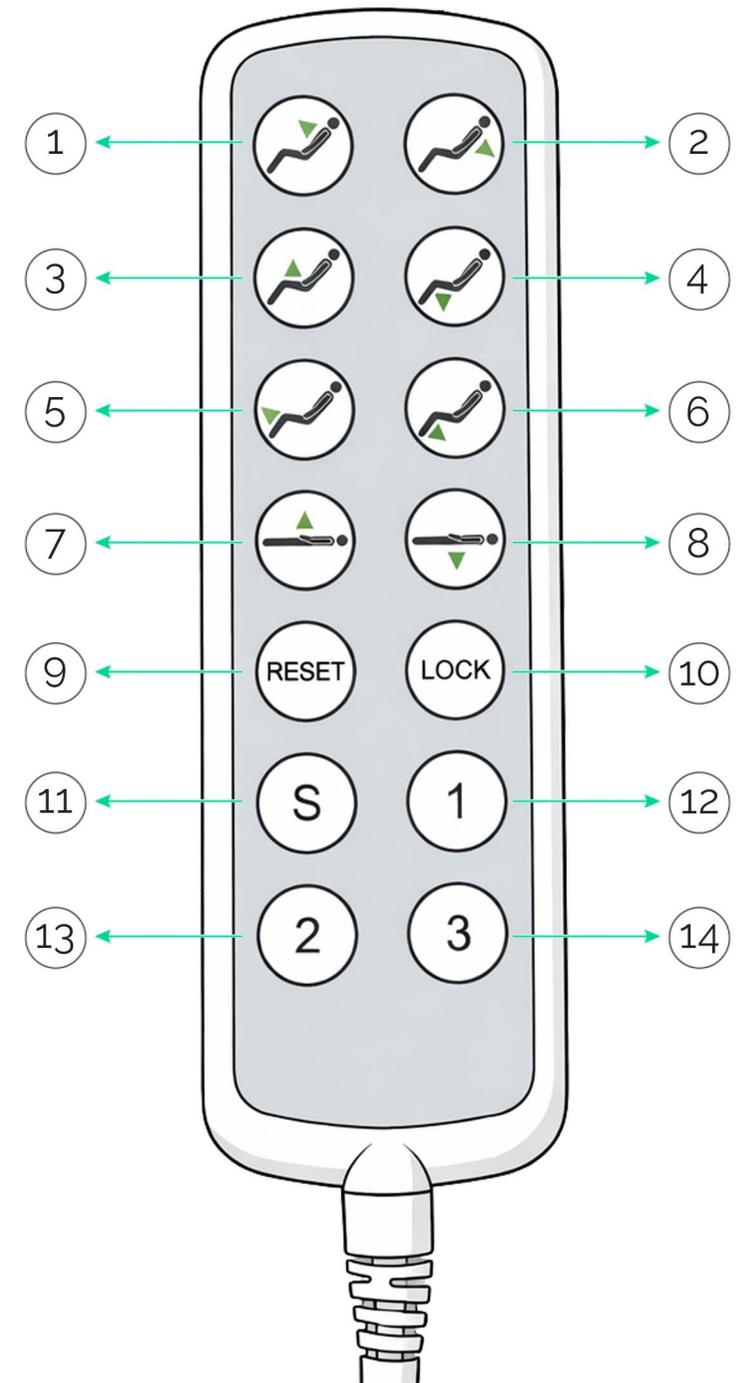

Installation Preparation: We recommend two (2) people for installation, as both individuals are required to lift and carefully remove the bed from the pallet onto the floor.


STEP 1

With the help of another person, carefully lift and remove the bed from the pallet and place it onto a flat, stable surface.

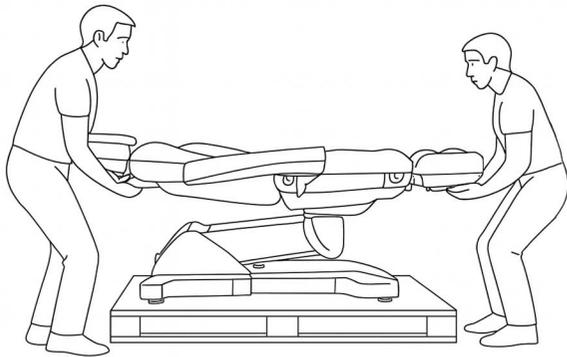



STEP 2

Remove the leg extension (B) from its packaging, loosen the knob under the legrest, insert the extension, and tighten the knob securely to lock it in place.

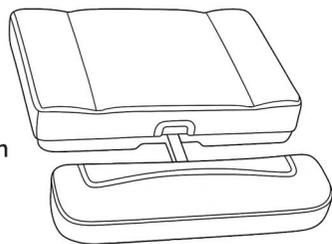

STEP 3

Remove the head rest (A) from its packaging, loosen the knob under the head rest, insert the head rest, and tighten the knob securely to lock it in place. Then insert the head pillow.

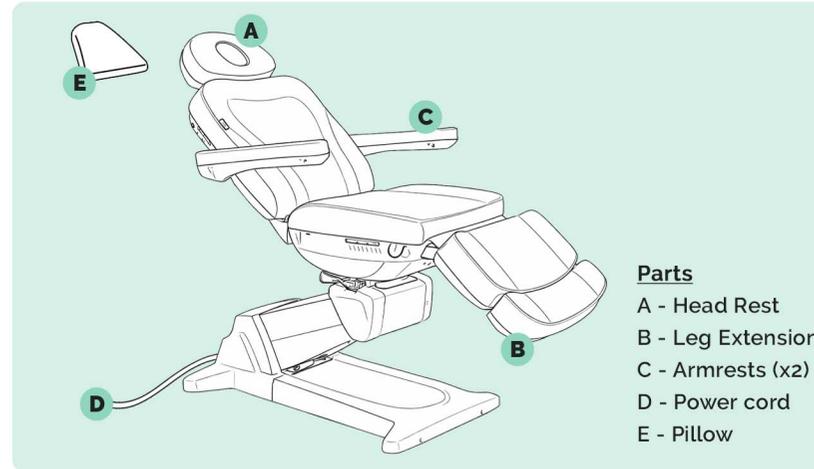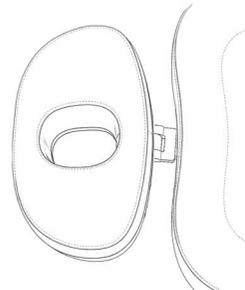


Parts

- A - Head Rest
- B - Leg Extension
- C - Armrests (x2)
- D - Power cord
- E - Pillow


STEP 4

Insert each armrest (C) by guiding the pole or bolt into the slot (D) on both sides of the chair. Ensure the armrests are facing upwards, as shown in the image below. Push firmly until you feel the armrests lock into place, confirming they are securely installed on each side.

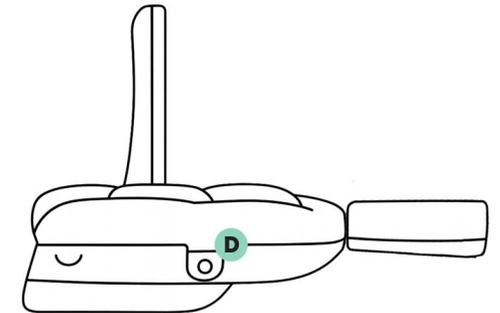

STEP 5

Plug the power cable into a standard wall socket, ensuring it is fully inserted. Make sure the power switch on the bed and the wall socket are both turned on before operating the bed.

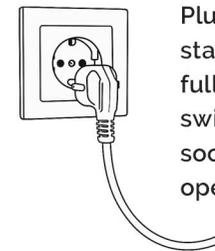

SAFETY WARNING: Do not allow children to operate this product. Always ensure that customers enter and exit the bed only from the middle section. Failure to follow these instructions may result in serious injury and damage to the bed.

Upholstery Care Guide

As a professional esthetician, investing in a high-quality electric facial bed can significantly enhance the comfort and effectiveness of your treatments. To maintain that level of performance and ensure long-term reliability, regular maintenance and proper care are essential. Like any professional equipment, your facial bed requires routine attention to stay in optimal working condition—ensuring safety, smooth functionality, and a consistently elevated experience for your clients.

General Care Tips

Maintain a stable temperature:

- Cold can lead to vinyl cracking
- Heat may cause permanent stretching

Keep out of direct sunlight for long periods to prevent fading and damage. Regular housekeeping helps preserve comfort and appearance.



Cleaning Instructions

Clean frequently to remove body oils that can harden vinyl

Use a colorfast cloth dampened with lukewarm water and mild, alkaline-free soap

Avoid soaking the material

Wipe off any soap with fresh water and a clean cloth

Dry thoroughly with a soft, dry cloth

For tougher grime, gently scrub with a soft brush



Avoid Using These Products

- Hot or boiling water
- Alcohol or hydrocarbon-based cleaners
- Acetone
- Bleach or harsh detergents
- Furniture polish
- Abrasive powders, pastes, or steel wool
- Silicone or petroleum-based products
- Solvent-based or abrasive cleaning agents

Routine Inspection

Perform periodic checks on:

- Hinges
- Fasteners
- Structural components

Ensure your chair remains safe, stable, and fully functional.

CE & RCM Certification

All electric components in our beauty beds are CE (Europe) & RCM (Australia & New Zealand) certified, meeting the strict safety, health, and environmental standards required in Australia, the EU, and the UK.



WARRANTY PERIOD

STRUCTURE: **5 YEARS** | VINYL: **2 YEARS** | MOTORS: **2 YEARS**

For more information about our warranty, please refer to our website.

DSSE

Australia:

www.dsse.com.au

United Kingdom:

www.dssebeauty.co.uk

Europe:

www.dsseworld.com